

Update TouchChat and TouchChat HD application software

The App Store® icon should show when updates are available to download on the device.

When there is an update available, the app store icon will display a number on the right hand corner of the icon.


1. Tap on the App Store® icon
2. Tap on Updates on the bottom right of the screen
3. The apps that require updating will show up
4. Tap either update on the application or you can tap update all in the upper right hand corner
5. Enter in the Apple® ID that you purchased TouchChat or TouchChat HD with previously
6. The update will start downloading

Note: if using a Dedicated System, see article, "Connecting to a Computer for the First Time"


To perform an update of the TouchChat app in older versions of iTunes:

1. Connect your device to your PC using the USB cable included with the device.
2. Launch iTunes® on your PC


3. Go to the Apps section of iTunes®

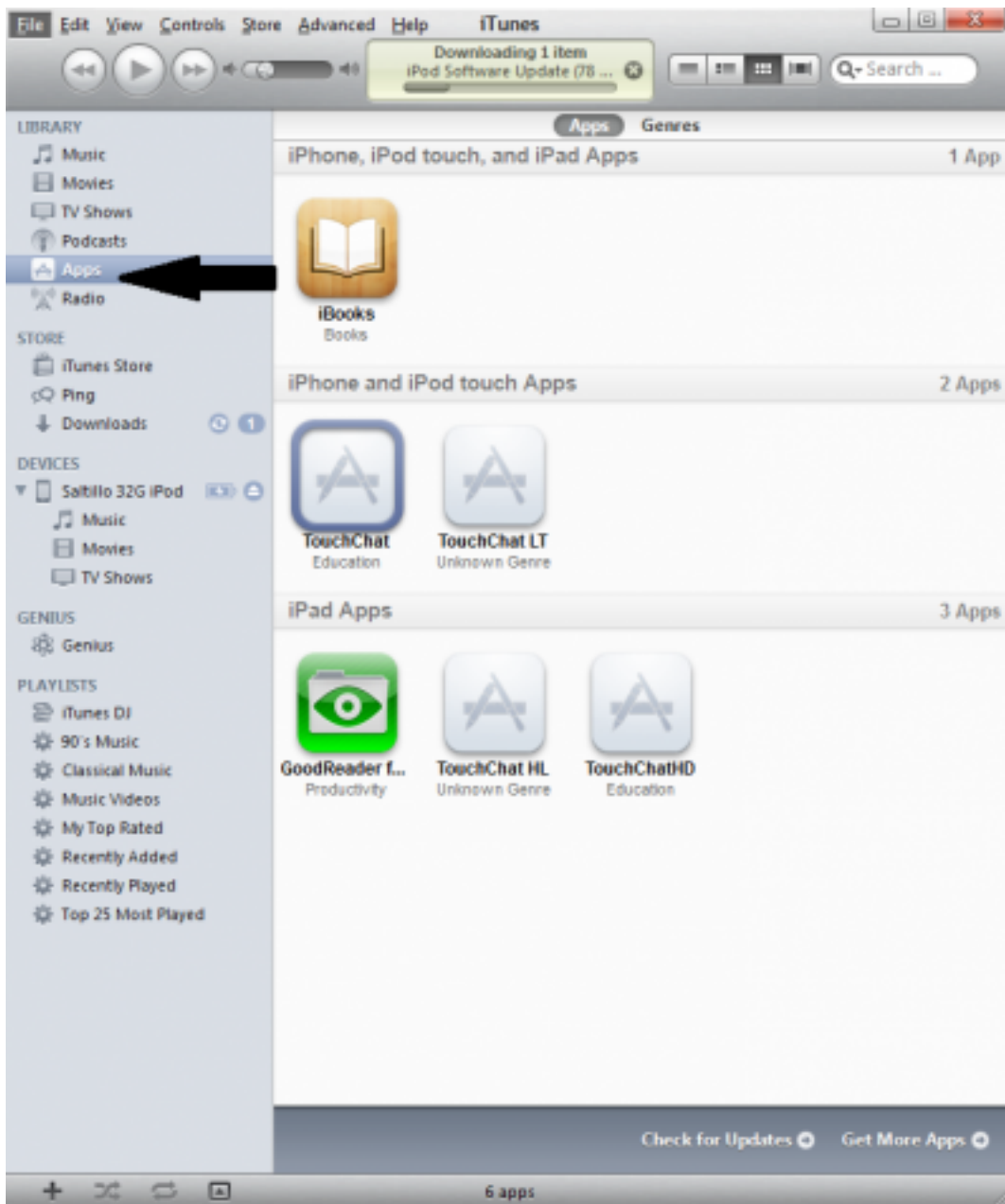

4. In the bottom right of the screen, there is an option to “Check for Updates”

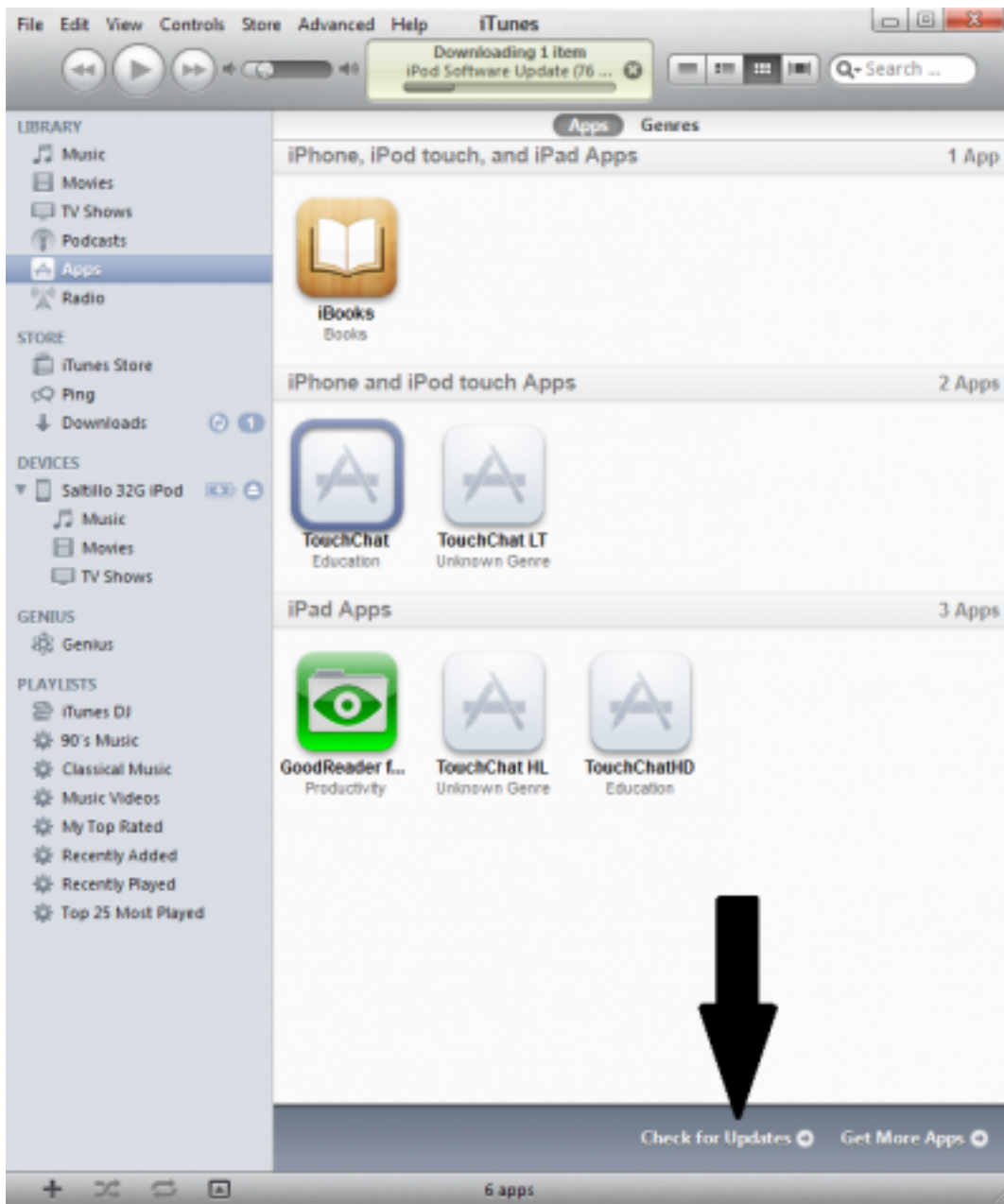

5. Enter in the Apple® ID that you purchased TouchChat or TouchChat HD with previously

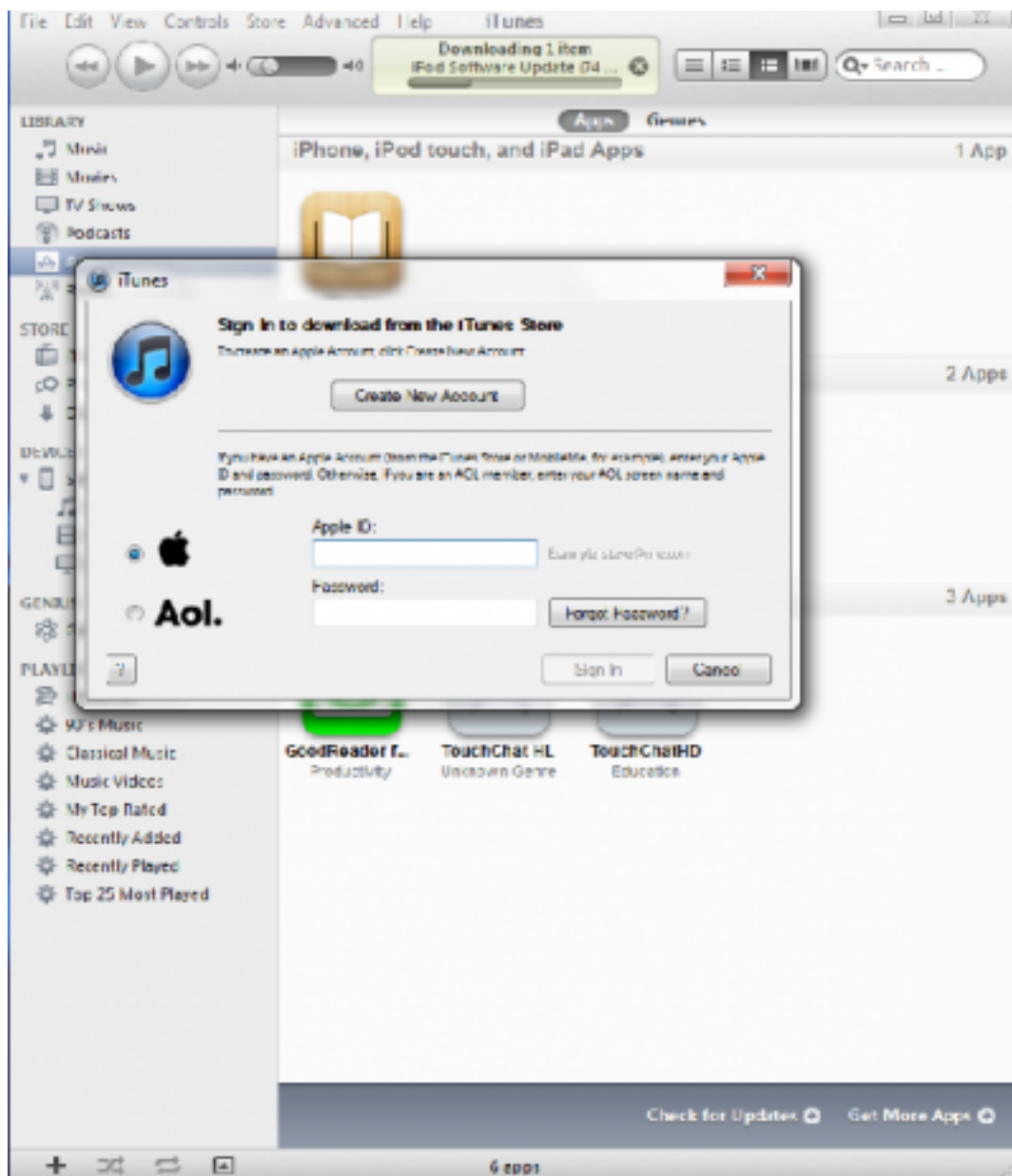



6. Update should start

Check the version number of TouchChat:

1. From the home page of the vocabulary file tap Vocab in the left corner

Vocab  Menu

Questions 	People 	Actions 	Describe 	Things 
Chat 	I want 	I don't want 	I need 	Clear 
Keyboard 	Core Words 	School 	My News 	My Scenes 

2. tap on Choose New Vocab



3. the version is at the bottom of the screen



Edit **Vocab File** **Menu**

	hello	v1.0
	mc	v1.0
	MultiChat 15 15 location, sentences, phrases, individual words, story-telling and visual scenes, emerging literacy	v1.0
	part	v1.0
	place	v1.0
	Primary Landscape 18 location, words, phrases and sentences, all symbol based for school or at home	v1.0
	Primary Portrait 16 location, words, phrases and sentences, all symbol based for school or at home	v1.0
	ret	v1.0
	Spelling QWERTY Keyboard layout with Word Prediction	v1.0
	VocabPC Landscape 12 location, carrier phrases, interactive sentences, activity vocabulary & naming words, developmental delays	v1.0
	VocabPC Portrait 12 location, carrier phrases, interactive sentences, activity vocabulary & naming words, developmental delays	v1.0
	zoo time	v1.0

